















Symbol	Title	Description
	Manufacturer	Indicates the medical device manufacturer
	Authorized representative in the European Community/ European Union	Indicates the authorized representative in the European Community/European Union
	Use-by-date	Indicates the date after which the <i>medical device</i> is not to be used.
	Batch code	Indicates the manufacturer's batch code so that the batch of lot can be identified.
	Catalogue number	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Temperature limit	Indicates the temperature limits to which the medical device can be safely exposed.
	Do not re-use	Indicates a medical device that is intended for one single use only
	Consult instructions for use or consult electronic instructions for use	Indicates the need for the user to consult the <i>instructions for use</i>
	Medical device	Indicates the item is a medical device.
	Unique device identifier	Indicates a carrier that contains unique device identifier information.

Diagram	Description
	Follow guidance in agreed risk assessment policy regarding the use of PPE.
	Refer to information completed within the 'opened on' and 'dispose by' section.
	Remove / dispense one wipe from the pack.
	Use a fresh wipe if your wipe becomes soiled or dry.
	Close the pack to protect the unused wipes.
	Use one wipe to disinfect. Wipe the surface in an S shape moving from clean to dirty. Use the wipe flat not scrunched. Do not go over the same area twice with the same wipe. If the surface is visibly soiled, more than one wipe may be required to clean the surface initially.
	Discard used wipes in the appropriate waste bin following local agreed guidelines.
	Allow the surface to dry naturally in line with stated contact time.
	Our tubs/canisters are made from HDPE with PP lids, these are fully recyclable
	Our buckets/lids are made from PP, these are fully recyclable
	Our flow wrap packs are made from laminate film and are not currently recyclable
	Do not flush
	Do not macerate

Diagram	Description
	<p>If in eyes, please rinse with water</p>
	<p>Please keep out of reach of children</p>
	<p><b>Flammable.</b> The product contains Hazard and Precautionary Phrases in accordance with CLP Regulation (EC) No 1272/2008. The EU CLP Regulation adopts the United Nations' Globally Harmonized System (GHS) of the classification and labelling of chemicals. Please read the accompanying H phrases and P phrases on the product packaging to understand the product safety requirements.</p>
	<p><b>Warning.</b> The product contains Hazard and Precautionary Phrases in accordance with CLP Regulation (EC) No 1272/2008. The EU CLP Regulation adopts the United Nations' Globally Harmonized System (GHS) of the classification and labelling of chemicals. Please read the accompanying H phrases and P phrases on the product packaging to understand the product safety requirements.</p>